

"It healed my knees within less than a week. Once he gave me the GlideWear, it was like night & day."

Pauline Wiirre
Bilateral transtibial amputee

"When I'm using it (GlideWear), it totally eliminates my need to take off the limb for periods of rest."

LCpl Mar Seifert, USMC, Retired
Transtibial amputee

"I love the stuff. I can't talk about it enough."

Dohn Fadden
Transtibial amputee

"Simple solutions like the use of a small GlideWear patch can allow amputees to gain the full benefits of their advanced prosthetic componentry by allowing them to spend additional time in their prosthesis without the incidence of skin breakdown."

Charles W. Kuffel
MSM, CPO, FAAOP
President & Clinical Director,
Arise Orthotics & Prosthetics



Earn Distance Learning
Continuing Ed. Credits

1.5 Credits

Clinical Educational Content

Advanced information for healthcare providers and end-users seeking more information about the clinical benefits and specific indications for recommending and using the GlideWear Prosthetic Liner Patch.

For on-line coursework and
accredited post-presentation quiz,
visit <http://goo.gl/xdzxy9>



A PART OF THE BECKER/TAMARACK O&P EDUCATION SERIES

GLIDE WEAR[®] by Tamarack

For more information, visit
www.GlideWear.com

Phone: 763-795-0057

Toll-free: 866-795-0057

E-mail: info@tamarackhti.com

 **Tamarack Habilitation Technologies, Inc.**
1670-94th Lane NE, Minneapolis, MN 55449 USA
Tel: +1 (763) 795-0057
E-mail: info@tamarackhti.com

 **QNET BV**
Kantstraat 19
NL-5076 NP Haaren
The Netherlands

GLIDE WEAR[®] by Tamarack

PROSTHETIC LINER PATCH



For **immediate relief**
and **lasting protection**
from skin breakdown

Indications

The GlideWear Prosthetic Liner Patch is **indicated for lower limb amputees who experience painful skin irritation and breakdown** on their residual limb from daily use of a prosthesis.

When used in conjunction with a prosthetic liner, GlideWear provides immediate relief and lasting protection from skin breakdown caused by friction & shear forces between skin tissue, bony prominences, and the prosthesis.

Features & Benefits

Easy to Use

Place GlideWear® Prosthetic Liner Patch directly against the area of concern, before donning liner over the top of the patch and limb. Use a clean patch daily to protect against future breakdown.

Stays In Place All Day Long

GlideWear® Prosthetic Liner Patches are held in place by the gel liner or prosthetic socket; ensuring uninterrupted skin protection all day long.

Reduces Shear

Patented dual-layer GlideWear® technology absorbs friction and related shear forces at the skin-prosthetic liner interface.

Durable & Reusable

Machine or hand wash frequently to keep clean and maintain effectiveness.

Everyone Saves

Amputees save money & energy otherwise spent purchasing and applying messy creams, powders and bandages.

Healthcare professionals reduce non-billable time and office expense associated with socket modifications and wound care efforts.

Specifications

Construction:
85% Nylon
15% Spandex
Does not contain latex.

Available Sizes:

Small
2.5 in. x 4.75 in.
(6 cm x 12 cm)
Part No. GW-PRO-2SB


Large
4.5 in. x 7.5 in.
(11 cm x 19 cm)
Part No. GW-PRO-2LB

Circle
4 in. dia. (10 cm)
Part No. GW-PRO-2CB



MADE IN MINNESOTA + U.S.A. +

CARE INSTRUCTIONS

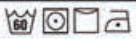

Wash with mild detergent (≤ 140° F). Tumble or air dry.

Application & Placement Tips

Contact your Prosthetist to determine if the GlideWear® Prosthetic Liner Patch is appropriate for your skin care needs.

Place the GlideWear® Prosthetic Liner Patch **between the prosthetic liner and skin** – adjacent to the area of concern – while donning the liner.

Do not place directly over an open wound.

For placement in distal applications, lay the GlideWear Prosthetic Liner Patch over the rolled edge of the liner before donning the liner onto the limb.

Use a clean and dry patch daily to ensure lasting protection from discomfort, redness, and skin breakdown.

GLIDE WEAR[®] by Tamarack

PROSTHETIC LINER PATCH

For more information, visit
www.GlideWear.com

Complete case study photos and notes available online

