# ShearBan<sup>®</sup>

by Tamarack<sup>®</sup>

## Installation Instructions

- 1) Clean application surface as necessary to remove dirt & moisture.
- 2) Cut desired shape/size of ShearBan with sharp scissors, ensuring the ShearBan patch will extend beyond the area of required coverage.
- 3) Round all corners of the ShearBan patch to prevent peeling after installation.
- 4) Apply ShearBan directly to the footwear, insole, orthosis or prosthesis – not against the skin. Press firmly around entire patch surface to ensure adhesion.

## For Additional Application Tips & Techniques

Visit [www.shearban.com](http://www.shearban.com) and click on **ShearBan Product Guide** under Downloads

### User Notice:

Tamarack does not advocate ShearBan as a substitute for proper contouring of supportive devices and is not responsible for undesirable results due to improper friction management practices.

ShearBan is a friction and shear reducing interface material serving as a prophylactic means to prevent, heal, and avoid recurrence of tissue trauma in orthotic and prosthetic applications. The self-adhesive back is to be applied to body support or containment surfaces, not to the human skin.

ShearBan's low friction surface is comprised of a PTFE film which is intended to bear against an item of fabric clothing or directly against healthy, intact skin. It should not be used to directly cover or bear against open wounds or abraded skin without an additional wound care dressing between ShearBan and the skin.


ShearBan is dermatologically inert and listed with the FDA under the classification # 890.3025.

For best results, store unused ShearBan in the bag provided.

Manufactured in the USA by

**Tamarack<sup>®</sup>**  
HABILITATION TECHNOLOGIES, INC.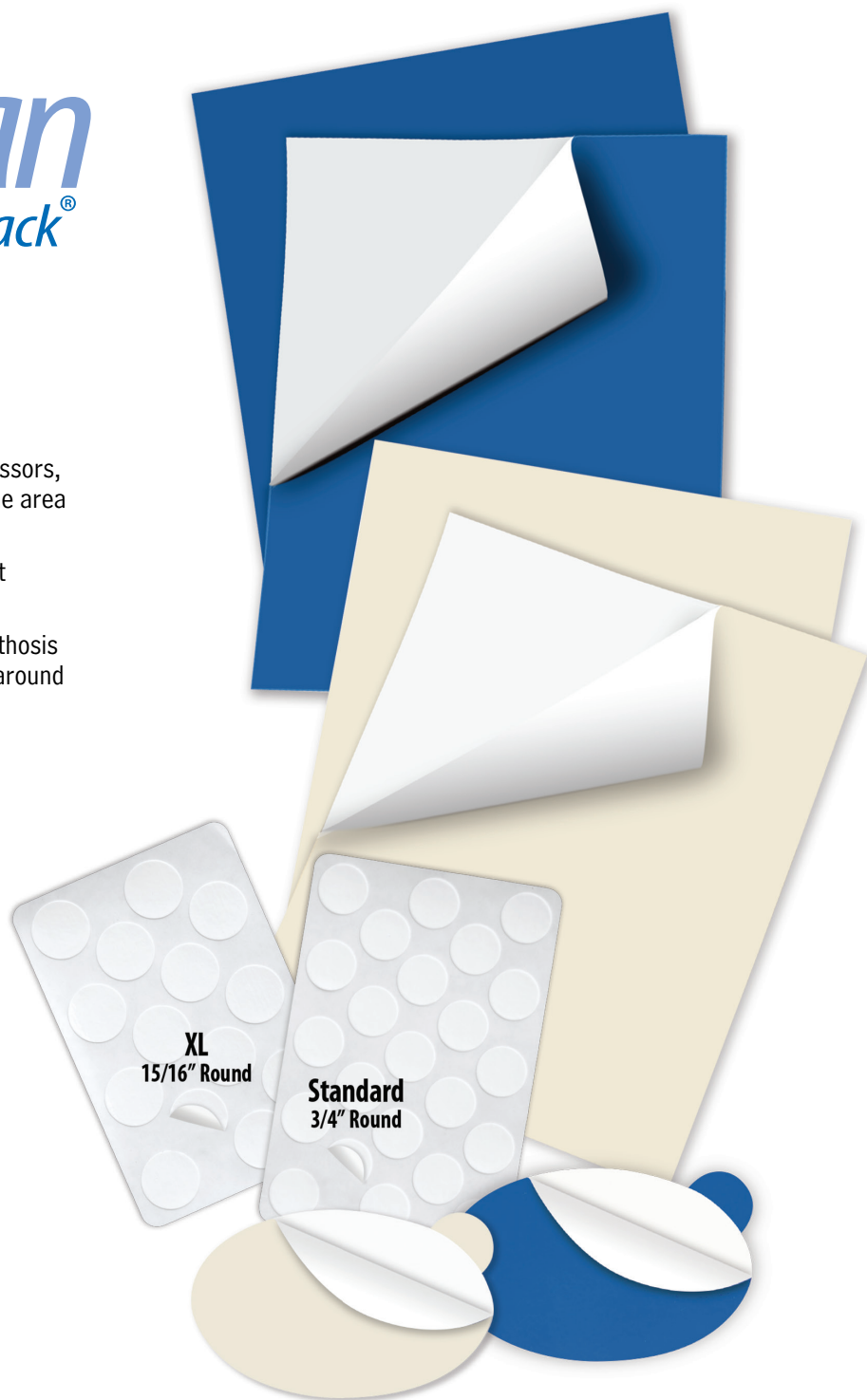

### Available in Single & Five-Sheet Packs and Pre-Cut Ovals

PART #	DESCRIPTION	QUANTITY
749S-BEIGE	8" x 12" ShearBan <sup>®</sup> Sheet (beige)	1 Sheet
749S-BLUE	8" x 12" ShearBan <sup>®</sup> Sheet (blue)	1 Sheet
749-BEIGE	8" x 12" ShearBan <sup>®</sup> Sheets (beige)	5 Sheets
749-BLUE	8" x 12" ShearBan <sup>®</sup> Sheets (blue)	5 Sheets
749-OV12-BL	1.75" x 2.75" ShearBan <sup>®</sup> Pre-Cut Ovals (blue)	12 Ovals
749-OV12-BE	1.75" x 2.75" ShearBan <sup>®</sup> Pre-Cut Ovals (beige)	12 Ovals
749-7	3/4" ShearBan <sup>®</sup> Rivet Cover Patches (beige)	138 pcs.
749-7-XL	15/16" ShearBan <sup>®</sup> Rivet Cover Patches (beige)	86 pcs.

(Web) [www.tamarackhti.com](http://www.tamarackhti.com) • (Email) [info@tamarackhti.com](mailto:info@tamarackhti.com) • (Phone) 1-763-795-0057

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